User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

139

EN

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Verze: B, 2022-10

All rights reserved, including translation.

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1 Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this toilet chair and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

2 Your product

- | | |
|------------------|--------------------------|
| 1. Backrest | 7. Footplate |
| 2. Armrest | 8. Castor with brake |
| 3. Arm pad | 9. Castor without brake |
| 4. Seat | 10. Frame |
| 5. Toilet bucket | 11. Identification plate |
| 6. Footrest | |

3 Before use

3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Frame (incl. commode seat, seat cushion, armrests, 4 steering castors, 2 fixing screws for backrest)
- Backrest
- Footrests with footplates
- Toilet bucket with cover
- Instruction manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

3.2 Intended use

Indications and contraindications: Mobile, reusable toilet commode for indoor use, intended for users who suffer from incontinence or mobility problems. Should not be used for taking a shower, for storage, as a ladder, or a wheelchair.

The toilet chair is designed and produced solely to be used as toilet aid for one person with a maximum weight of 120 kg.

3.3 Safety instructions

CAUTION

Risk of injuries

Please read and follow the instructions in this manual. Otherwise you may get injured, or your toilet chair may get damaged.

- Do not use as a wheelchair.
- Do not lift the chair by the armrests or footrests.
- Do not stand on the footrests, the toilet chair could tip over.
- When setting up the chair and when it is transported, carefully guard against bruising and shearing.
- Do not grasp sideways further than the length of your arm allows.
- Do not lean too far forwards.
- Remove the bucket before transporting the chair.
- Empty the bucket every time after use.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

4 Assembly

4.1 Frame

Position the base frame in front of you facing forward. Hold the backrest on the side at the armrest screws (in doing this, the armrests should hang down) and guide the back tubing into the tube openings provided on the base frame. Turn the locking screws until they are tight.

The locking screws must be released, otherwise it may not be possible to insert the back tubing properly.

Check that the back tubes are not jammed or skewed and that they are pushed in fully.

4.2 Armrests

Place the armrests in the receivers provided on the base frame, by rotating the armrests to the front from below on their screw fixings.

4.3 Footrests

The footrest receivers are located on the front frame tubes of the toilet chair.

1. Fold the footplates open.
2. Place the footrests on the side into the two guide bolts and swing inwards until they lock in place. If this does not happen immediately, try again while gently pressing the retaining mechanism (inner side of footrest hanger).
3. Mount the calf strap over the hooks.

4.4 Seat cushion

To place the seat cushion in the correct position, fit the guide pin between the commode seat and its hand grip (at the back).

4.5 Toilet bucket

The toilet bucket can be pushed below the seat from the back, with or without its cover.

5 Usage

5.1 Brakes

CAUTION

Risk of injury

Do not operate the brakes with your hands.

Before getting into or off the toilet chair, the parking brakes must be applied to the rear castors. The brakes are applied by pushing down the silver brake lever with your foot.

5.2 Toilet bucket

- Remove the bucket before transporting the chair.
- Remove and empty the bucket every time after use.
- Pour some water in the upper edge of the bucket. The bucket is then sealed airtight against bad odours when the lid is closed.

6 Maintenance

6.1 Cleaning

The seat cushions, arm cushions, frame and bucket can be wiped using domestic cleaning agents and a damp cloth. Do not use a brush or any other type of scourer.

Chrome parts should be wiped periodically with a dry cloth. Use a mild chrome polish on stubborn dirt.

Treatment with hot jets and high pressure equipment is not permitted.

6.2 Disinfection

Use a soft, damp cloth and lukewarm water with a mild household disinfectant to clean the toilet chair.

Toilet buckets and cushioned seats are not meant to be treated hygienically. In case of serious or permanent contamination, consider replacing the toilet bucket and cushioned seats.

6.3 Inspection/Service/Repairs

The expected service life of your toilet chair is 5 years. The lifespan of the toilet chair is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the toilet chair, it should be inspected on the following points:

- Check the frame, armrests, footrests and castors for deformation and reduced functioning.
- Visual inspection of damage to the paintwork (danger of corrosion)
- Check the condition of the cushions (tears, brittleness, soaking)
- Bucket sealing (not in the case of re-using)
- Completeness of consignment ?
- Instruction manual available?

Your toilet chair is constructed of steel and plastic components. We recommend to have your toilet chair inspected at least once a year, but the service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

6.4 Reuse

Before each reuse, have the toilet chair disinfected, inspected and serviced according to the instructions in this chapter.

6.5 Disposal at end of life

At end of life, you need to dispose your toilet chair according to the local environmental legislation. The best way to do so, is to disassemble the toilet chair to facilitate the transport of recyclable parts.

7 Technical details

Brand	Vermeiren
Type	Toilet chair
Model	139

<i>Description</i>	<i>Dimensions</i>
Max. load	120 kg
Total height	950 mm
Seat height	492 mm (without seat cushion) 528 mm (with seat cushion)
Overall width	540 mm
Seat width	420 mm
Backrest height	425 mm (without seat cushion) 390 mm (with seat cushion)
Overall length	820 mm
Seat depth	420 mm
Dismantled length	550 mm
Dismantled height	520 mm
Armrest to seat distance	265 mm (without seat cushion) 225 mm (with seat cushion)
Front location of armrest structure	395 mm
Weight	ca. 13 kg
Dismantled weight	Frame: 6,85 kg Backrest: 2,4 kg Footrests: 1,75 kg (Pair) Seat cushion: 1,15 kg Bucket: 0,7 kg
Castors	125 mm (2 with brakes, 2 without brakes)
Volume toilet bucket	5 l
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.	

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Vermeiren GROUP
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com