# VERMEIREN

## 9060

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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## 1 Preface

First of all we wish to thank you for the trust you placed in us by selecting a VERMEIREN product.

Before using the toilet chair, read the instruction manual carefully: it will familiarise you with this product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### **Important note**

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

### **Information available**

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

## 2 Before use

### 2.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Toilet chair + cushioned seat
- Cushioned lid
- Toilet bucket with cover
- Instruction manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 2.2 Intended use

Mobile, re-usable toilet commode. Should not be used for taking a shower, for storage, as a ladder, or over built-in toilets.

### 2.3 Safety instructions

- When setting up the chair and when it is transported, carefully guard against bruising and shearing – danger of injury.
- Do not grasp sideways further than the length of your arm allows – danger of injury.
- Before use, check whether all four legs stand evenly on the floor.
- Do not lean too far forwards -- danger of tipping over.
- Remove the bucket before transporting the chair.
- Repairs and disinfection should only be done by authorised persons.
- Empty the bucket every time after use.

## 3 Assembly

The toilet chair is delivered fully assembled. The bucket can now be pushed onto the guide rails below the seat from the left. When not in use the seat can be covered by the cushioned lid supplied.

## 4 Usage

- Remove the bucket before transporting the chair.
- Remove and empty the bucket every time after use.
- Pour some water in the upper edge of the bucket. The bucket is then sealed airtight against bad odours when the lid is closed.

## 5 Maintenance

### 5.1 Cleaning

The seat cushions, arm cushions, frame and bucket can be wiped by using domestic cleaning agents and a damp cloth. Do not use a brush or any other type of scourer.

Chrome parts should be wiped periodically with a dry cloth. Use a mild chrome polish on stubborn dirt.

Treatment with hot jets and high pressure equipment is not permitted.

## 5.2 Disinfection

All parts of the chair can be disinfected by scrubbing. In principle all surfaces of a system or a product have to be disinfected before handing it over to another user (re-use).

Only authorised technicians trained in the functioning and application of disinfectants should apply disinfectants.

Toilet buckets and cushioned seats are not meant to be treated hygienically.

## 5.3 Inspection/Service/Repairs

The toilet chair must be checked at least every second year by authorised persons/specialist dealers, and at least every time before resuming its use. The following parameters must be checked:

- Check the frame for plastic deformation and reduced functioning.
- Visual inspection of damage to the paintwork (danger of corrosion)
- Check the condition of the cushions (tears, brittleness, soaking)
- Bucket sealing (not in the case of re-using)
- Completeness of consignment ?
- Instruction manual available?

The service should only be confirmed in the "Service plan" if at least the above aspects have been checked. If your specialist dealer does not undertake the services which you require, kindly contact the manufacturer. We will gladly refer you to authorised specialist dealers in your vicinity.

Replacement parts can be obtained from your specialist dealer.

## 6 Technical details

<b>Brand</b>	<b>Vermeiren</b>
<b>Type</b>	<b>Toilet chair</b>
<b>Model</b>	<b>9060</b>

<i>Description</i>	<i>Dimensions</i>
<b>Max. load</b>	<b>120 kg</b>
Total height	800 mm
Seat height	490 mm
Overall width	550 mm
Seat width	420 mm
Distance between armrests	450 mm
Backrest height	350 mm
Overall depth	525 mm
Seat depth	450 mm
Armrest to seat distance	220 mm
Weight	ca. 7,2 kg

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

Vermeiren GROUP NV  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

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### All rights reserved, including translation.