
Raised toilet seat Tina

Our raised toilet seat was designed to raise the height of a toilet to a comfortable level, making it easier to sit down and get up.

1. Lift the seat and lid of the toilet.
2. Turn the adjustment knob counterclockwise to loosen the adjustment bracket. This will allow the raised toilet seat to fit onto the toilet bowl.
3. Place the raised toilet seat onto the bowl rim, hooking the rear flanges under the edge of the bowl. Check that the unit sits flat on the bowl rim. The knob must be at the front end of the raised toilet seat.
4. Center the raised toilet seat on the bowl rim and push it back as far as possible with your hands.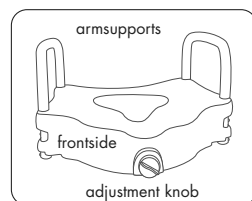

5. Turn the adjustment knob clockwise to tighten. Turn until resistance is felt. Turn a little bit more until secure. Turn only by hand - do NOT use a wrench.

6. If the top rear of the seat hits the toilet seat or lid, you must remove the toilet seat for proper fit.

Inserting the arm supports

Two arm supports came with your toilet seat. Press the two snap buttons simultaneously and slide the arm supports in the tubes. Make sure the snap buttons are locked properly in position.

This toilet seat is not intended for permanent installation. It should be loosened and retightened once a week to ensure proper fit.

! Use the arm supports only for assistance. Do not use the arm supports to support your full body weight.

Slowly lower yourself onto the raised toilet seat by bending your elbows, hips and knees if possible.

! Use only non-aggressive cleaning products. Rinse well and wipe with a non-abrasive cloth. Users weight: 130 kg

Subject to technical alterations. © 2016

VERMEIREN

Opening up new horizons in mobility

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Vermeiren Nederland B.V.