

User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

631



Content

Content	1
1 Preface	1
2 Your product	1
3 Before use	1
4 Assembly	2
5 Use of the lifting pole	2
6 Maintenance	3
7 Technical details	3

1 Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this product and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.


Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

2 Your product

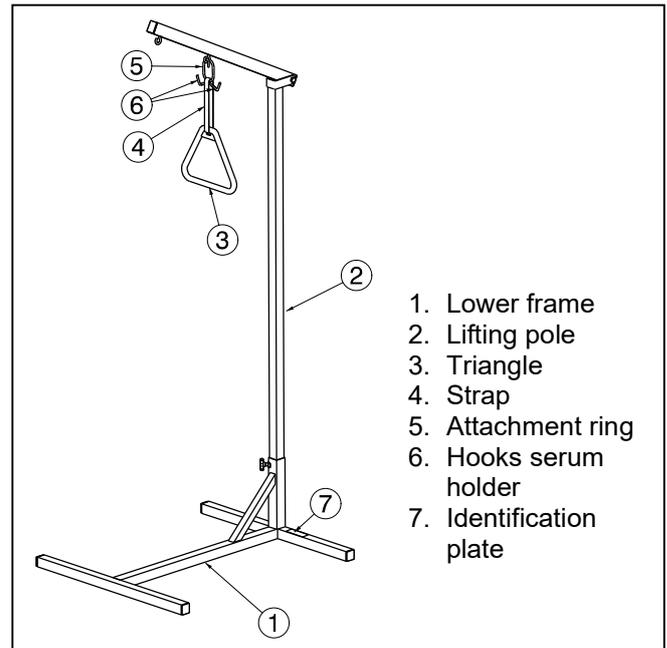

1. Lower frame
2. Lifting pole
3. Triangle
4. Strap
5. Attachment ring
6. Hooks serum holder
7. Identification plate

3 Before use

3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- lower frame
- lifting pole + triangle
- manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

3.2 Intended use

- The lifting pole shall be used by 1 person.
- The lifting pole shall be used indoor.
- The lifting pole shall not be used as toy for children.
- The lifting pole shall be used as a raising aid from a lying to a sitting position.
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

3.3 Safety instructions



CAUTION

Risk of injury and/or damage

- Do not exceed the maximum load (80 kg).
- Take note of the instructions for care and service. The manufacturer is not liable for

damage caused by improper servicing/care.

- Always lock the wheels of the bed whilst using the raising aid.
- Ensure correct assembly (see §4).
- Before use ensure that the strap sits correctly between both holders.
- Only place the raising aid at the head of the bed and on a level surface.
- Assembly and usage is only permitted under the supervision of qualified people.
- Do not use or try to straighten the raising aid if deformed.
- The device is not fire-resistant and should therefore not be placed close to fire or extreme heating.

3.4 Shipping and storage

The shipping and storage of the product should happen according to the technical specifications in §7. Make sure that your product is stored dry.

Provide sufficient covering or packaging to protect the product from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

Make sure no objects are stored on or against your product to prevent damage.

3.5 Symbols





Maximum weight



Type designation

4 Assembly

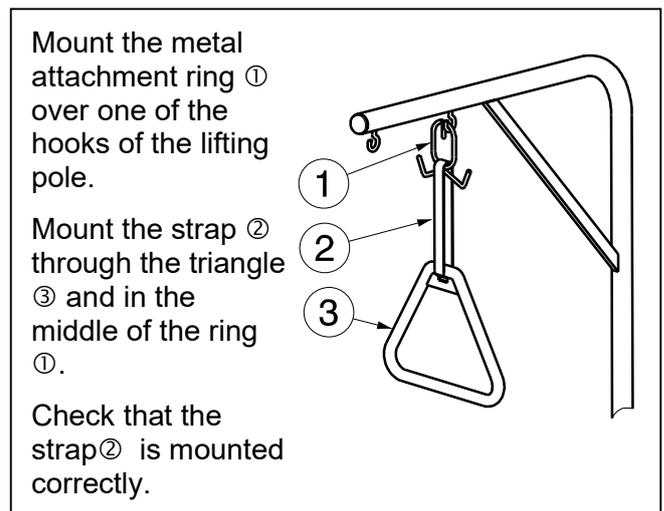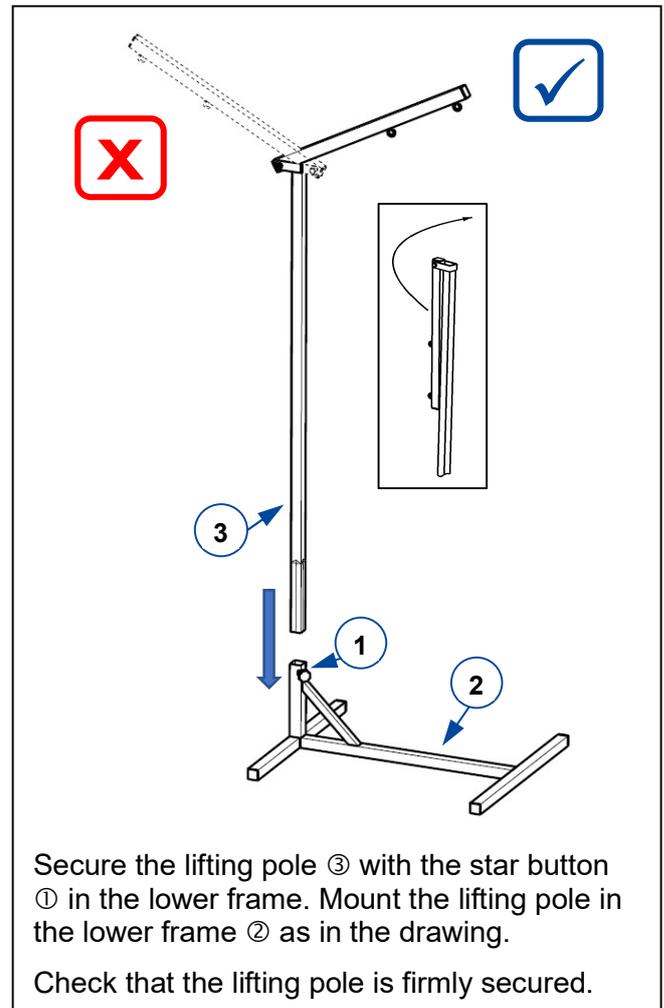

5 Use of the lifting pole

Place the lifting pole on the head side of the bed. Check that the underground is a level surface. Now the patient can grasp the triangle and pull himself up from lying position to sitting position.

6 Maintenance

6.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap. Avoid soaking. Do not use abrasive cleaning agents to clean.

6.2 Disinfection

 **CAUTION** Risk of damage

Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

Use a soft, damp cloth and lukewarm water with a mild household disinfectant.

6.3 Inspection / Service / Repairs

The expected service life of your lifting pole is 5 years. The lifespan is influenced by use, storage, maintenance, service and cleaning.

Before you resume use of the lifting pole, it should be inspected on the following points:

- Frame (deformation, stability)

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

6.4 Reuse


Before each reuse, have your product disinfected, inspected and serviced according to the instructions in §6.

6.5 Disposal at end of life

At end of life, you need to dispose of your product according to the local environmental legislation. The best way to do so, is to disassemble the product to facilitate the transport of recyclable parts.

7 Technical details

 **WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

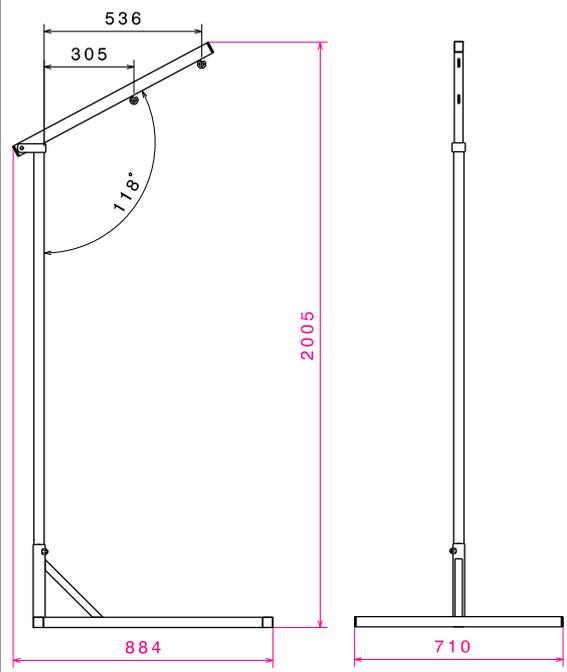Make	Vermeiren
Type	Lifting pole
Model	631
	
Total weight	11 kg (lifting pole: 5,15 kg lower frame: 5,6 kg, triangle + strap: 0,25 kg)
Maximum load	max. 80 kg
Storage temperature	+5°C - +41°C
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	



Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:



Vermeiren GROUP
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com

Instructions for specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: C, 2022-12

Basic UDI: 5415174181899631EF

All rights reserved, including translation.