VERMEIREN

691

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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Telescopic-Ramps: 691

First of all we wish to thank you for the trust you placed in us by selecting a VERMEIREN product.

Before using the side ramps, read the instruction manual carefully: it will familiarize you with this product. You will also be given hints about the care and service of your ramps. Please remember that observance of our instructions would keep your ramp in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, kindly consult your specialist dealer.

Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 2 Telescopic ramps
- Instruction manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- Contact the transporter
- Have a list made of any problems
- Contact your supplier

Technical data

	1-m ramp	1.6-m ramp
Length	1060 - 1930 mm Telescopic	1660 - 3130 mm Telescopic
Weight	14 kg (Pair)	20 kg (Pair)
Width	250 mm (Piece)	250 mm (Piece)
Track width	190 mm	190 mm
Edge	ca. 50 mm	ca. 50 mm
Load	max. 220 kg (Pair)	max. 220 kg (Pair)
Material	Aluminum	Aluminum

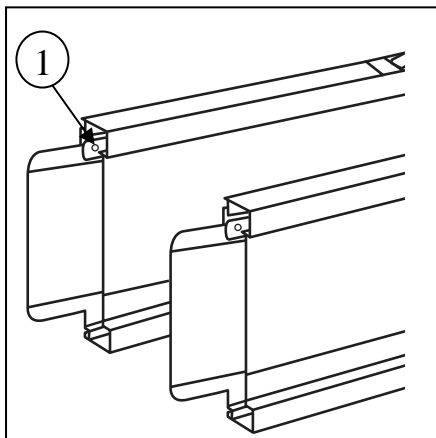
Usage

Short-term use for the transport of persons in a wheelchair to overcome differences in height of max. 10%.

Slide out / Adjustment

At the top of the ramp there is a locking plate ①. Move this slowly upwards, the telescopic ramps can now slide out or adjusted in length.


- 1-m ramp
Adjusting the length: 1060 mm – 1930 mm
- 1,6-m ramp
Adjusting the length: 1660 mm – 3130 mm

Assembly

The ramp is delivered in pre-assembled condition. Slide out the ramp until the desired height.

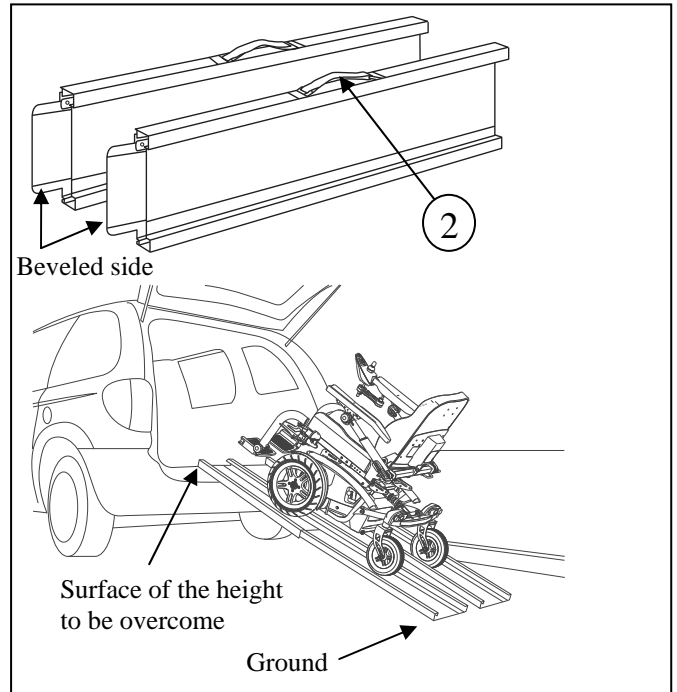For support please place the beveled side entirely on the surface of the height to be overcome (not the ground side).

Transport

To transport slight the ramps gently together. Make sure the locking plate is secured.

Each ramp is foreseen of a carrying handgrip ②. Please lift and transport your ramp with this handgrip.

Safety instructions

- Make sure the maximum load of your ramps is not exceeded (**max. 220 kg (Pair)**).
- Do not drive over the lateral edge → Risk of tipping over !
- When transported in a wheelchair and to be safe it is best that a second person help you with the transport over the ramp. → Risk of tipping over / falling!
- In the case of a passenger transport on the ramps you may not exceed a slope of 10% → Risk of tipping over / falling!
- Placing objects on the ramp is prohibited → Risk of tipping over / falling!
- Make sure when transporting the ramps that there are no objects or people in the transport area → Risk of injury!
- Place the ramps on flat surfaces.
- Take care when adjusting the length, that no objects or body parts are caught between the Telescopic rails → Risk of injury!
- Pay attention for the track width of the wheelchair. All four wheels are touching the ground during transport!
- No permanent installation!

Cleaning

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Observe the instructions of the cleaning liquid used. Do not use a cleaning agent which contains solvents, as this could alter or dissolve the plastic parts.

Repair / Maintenance

You should use only genuine VERMEIREN replacement parts. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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VERMEIREN GROUP
Vermeirenplein 1/15
B-2920 Kalmarhout
BELGIUM

Notes

A series of horizontal dotted lines for taking notes.

Notes

A series of horizontal dotted lines for taking notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.vermeiren.it
e-mail: info@vermeiren.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärtinger Strasse 4
A-4061 Pasching
Tel: +43(0)7229 64900
Fax: +43(0)7229 64900-90
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Eisenbahnstrasse 62
3645 Gwatt (Thun)
Tel: +41(0)33 335 14 75
Fax: +41(0)33 335 14 67
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Manufacturer

Vermeiren GROUP

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

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All rights reserved, including translation.