VERMEIREN

Iris

USER MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

EN

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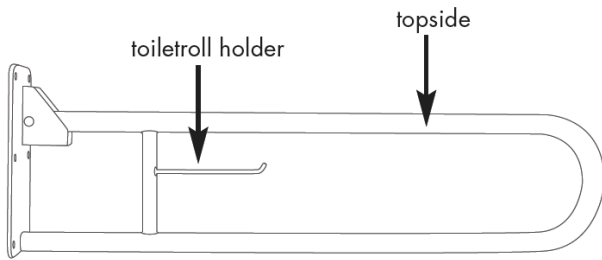
Foldable bar Iris

1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 1 foldable bar
- 6 screws
- 6 wall plugs
- Manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

2 Mounting the bathroom bar

CAUTION

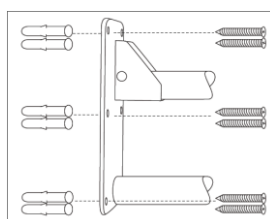
Risk of injuries

- Make sure that the bathroom bar is mounted securely to the wall. Check the fixation on a regular basis.
- Be sure that there are no electrical cables or other tubes in the mounting area of the bathroom bar.
- The plugs delivered with the product are suitable for concrete walls or full brick walls. For other types of walls, please inquire about the correct type of mounting equipment.

The foldable bar Iris is designed as a general support element. Additionally, a toilet roll holder is provided. They are useable everywhere, including humid environments. Use the installation materials which are delivered with Iris.

To install Iris:

1. Put the bathroom bar in the desired position.
2. Mark the fixation holes on the wall.
3. Drill holes in the wall.
4. Mount the plugs.
5. Place the bathroom bar against the wall.
6. Place the screws and fasten firmly.

If you have any doubts, do not hesitate to contact a professional to install Iris for you.

3 Using the bathroom bar

CAUTION

Risk of burns

- Be careful when using in extreme hot or cold environments (sunshine, extreme cold, saunas, etc.).
- If you notice any defects, refrain from using Iris.
- Avoid hanging heavy objects on the bar as it might cause damage.

The maximum pulling weight for the bathroom bar is 130 kg.

4 Maintenance

Iris can be cleaned and disinfected with normal household products. Avoid using abrasive cleaning products that can cause discoloration.

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

4.1 Expected lifespan

The bathroom bar is designed to have an average lifespan of 5 years.


5 Technical specifications

Brand	Vermeiren
Type	Bathroom bar
Model	Iris

Description	Dimensions
Maximum load	130 kg
Length	740 mm
Height	175 mm
Minimum mounting area (mounting plate)	230 x 100 mm
Storage temperature	+5°C - +41°C

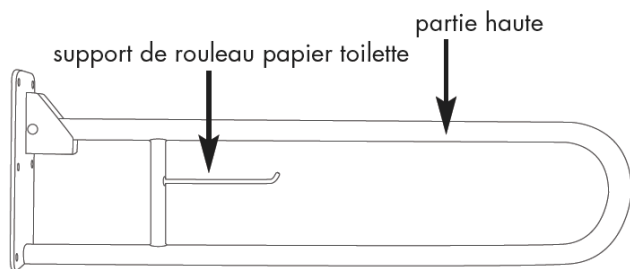We reserve the right to introduce technical changes.
 Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.

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All rights reserved, including translation.