


EP00218 / C

FRONT/REAR PANEL 996				Plan No. EP00218/ C
LUNA UL 2				
Scale: 1:8	Drafted	Verified	Confirmed	
Date	23/05/2016	18/10/2021	18/10/2021	
Initials	KR	DR	TK	

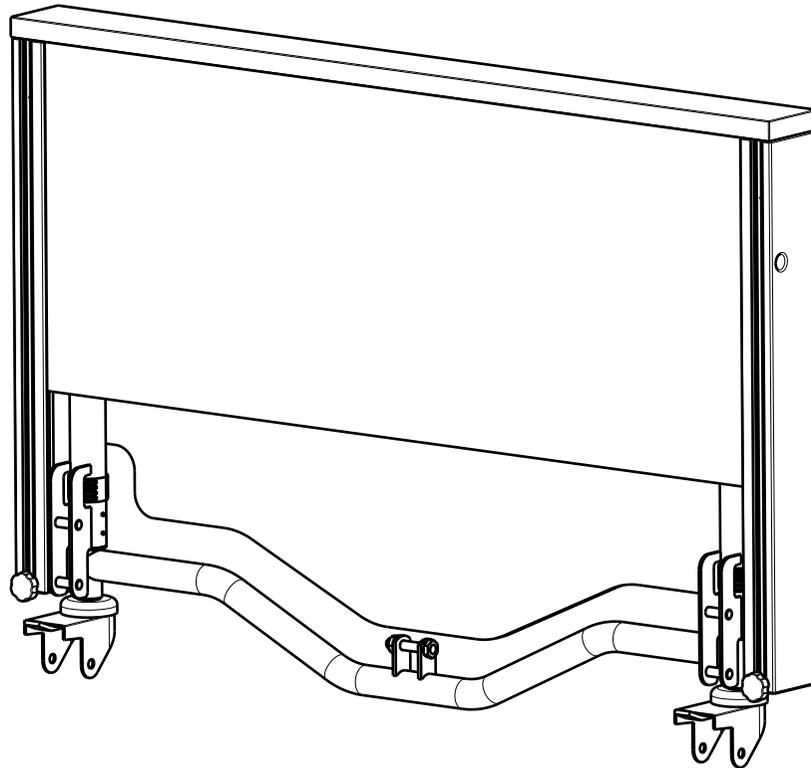

Material			
No.	Plan	Article	Quantity
1	YP05313	2707637	1
2	YP05314	2707737	1
3	XP03052	2730537	2
4	X002822	1995716	2
5	XP02239	1494600	2
6	---	1903974	2
7	XP03053	2996634	1
8	XP01399	2703900	1
9	X004564	1904875	2
10	---	1551600	1
11	---	1001572	2
12	---	1009318	2
13	---	1993130	6
14	---	1996200	1
15	---	1001683	1
16	StarKnob M6x15	1001577	2
17	---	1998658	2
18	StarlockD10	1009586	1
19	---	1001558	1
20	---	1007000	2
21	---	2990070	2
22	---	1008675	8
23	XP00791	2818600	1
24	X001497	1401900	-
25	Y016113	1904999	-
26	---	W620117	1

EP00218 / C

FRONT/REAR PANEL 996			Plan No.
LUNA UL 2			EP00218/ C
Scale: 1:8	Drafted	Verified	Confirmed
Date	23/05/2016	18/10/2021	18/10/2021
Initials	KR	DR	TK

