


Only to a depth of 32-36 .
 For depths of 36big-50 use EP00538

EP00595 / D

LEGREST BZ8 / TRIGO 30-36				Plan No.
				1/3 EP00595/ D
Scale:	1:3	Drafted	Verified	Confirmed
Date	16/07/2021	07/04/2023	07/04/2023	
Initials	AS	PM	AS	

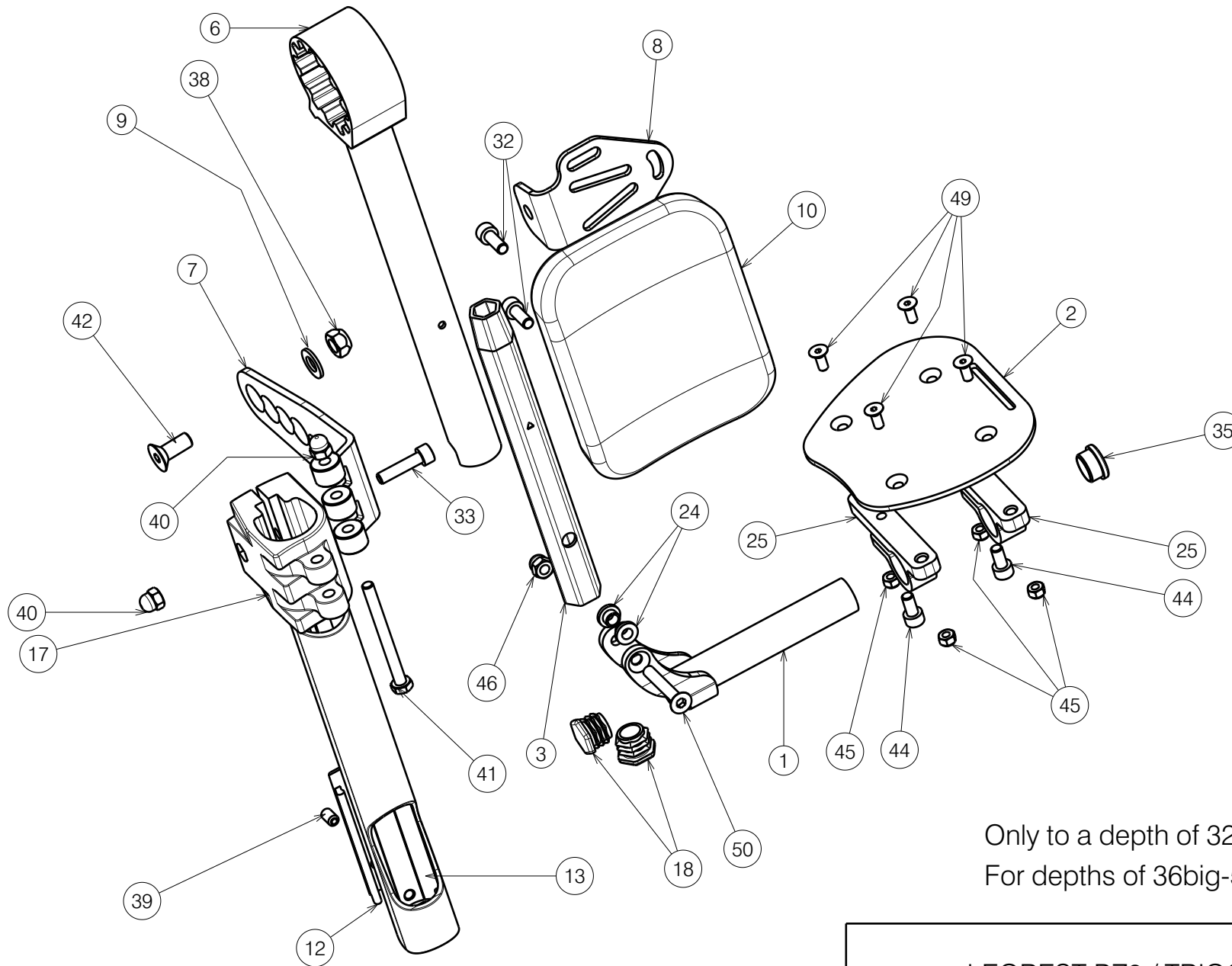



Only to a depth of 32-36 .

For depths of 36big-50 use EP00538

EP00595 / D

LEGREST BZ8 / TRIGO 30-36				Plan No.
				EP00595 / D
Scale: 1:3	Drafted	Verified	Confirmed	
Date	16/07/2021	07/04/2023	07/04/2023	
Initials	AS	PM	AS	

Material			
No.	Plan	Article	Quantity
1	YP06229(R)	2796365 (30-32)	1
	YP06126 (L)	2729965 (30-32)	1
	YP06254 (R)	2796665 (34-36)	1
	YP06893 (L)	2765665 (34-36)	1
2	YP06231(R)	2796454 (30-32)	1
	YP06232 (L)	2796554 (30-32)	1
	YP06256 (R)	2796754 (34-36)	1
	YP06257 (L)	2796854 (34-36)	1
3	YP06246(Short)	2796099	1
	YP09022(Long)	2631699	1
4	YP08517(R)	2618654	1
	YP08518(L)	2618754	1
5	YP09150(R)	2636454	1
	YP09151(L)	2636554	1
6	YP09153	2636654	1
7	YP09182	2637754	1
8	YP09164	2636854	1
9	X001119	1001554	1
10	X003779	1900228	1
11	X004423	1717300	1
12	X004658	1904968	1
13	X004666	1905731	1
14	X004690	1905735	1
15	X004713	1905734	1
16	X005880	1915832	2
17	X004739	1905739	1
18	X004774	1906056	2
19	X004795	1905970	2
20	X005028	G103900	1
21	X005047	1908420	1
22	X005055	1917169	1
23	X005046	G103800	1

Material			
No.	Plan	Article	Quantity
24	XP01127	2990164	2
25	XP01504	GP07699	2
26	XP01512	2993454	1
27	XP01523	GP08000	1
28	XP01546	2993452	1
29	XP01628	GP09100	1
30	XP02030	GP14499	1
31	XP03096	2996674	1
32	M6x16 DIN912	1001492	2
33	M6x25 DIN912	1001494	1
34	Rivet D4x14	1001584	4
35	Cap D19	1001602	1
36	Cap D25	1001608	1
37	M5 DIN985	1001679	1
38	M8 DIN985	1001682	1
39	M6x12 DIN916	1005949	1
40	M6	1008492	2
41	M6x65 DIN931	1991860	1
42	M8x20	1994677	1
43	M5x10 DIN7991	1999977	2
44	M6x12 DIN912	2994005	2
45	M5 DIN985	2994006	4
46	M6 DIN986	2994059	1
47	M6x25 ISO7380	2994062	1
48	D3.5x9.5	2994075	2
49	M5x12 DIN7991	2994076	4
50	M6x40 DIN7991	2994123	1
51	G0409	G0409	1

Only to a depth of 32-36 .

For depths of 36big-50 use EP00538

EP00595 / D

LEGREST BZ8 / TRIGO 30-36				Plan No.
3/3				EP00595/ D
Scale: 1:3	Drafted	Verified	Confirmed	
Date	16/07/2021	07/04/2023	07/04/2023	
Initials	AS	PM	AS	